

Know Before You Go

Quick Tip Guide for InsuJet™, a needle-free injection system for insulin administration



General

- The InsuJet™ is suitable for people with diabetes using U100 insulin. Please note that it is advised to involve your Diabetes Care Health Provider if you're considering using the InsuJet™ system.

Administration

- The nozzle can be used for 56 injections or 14 days, whichever comes first. Replace the comfort ring each time you replace the nozzle.
- The adapter requires one-time installation with an insulin vial, insulin cartridge, or insulin pen. Re-use until the insulin is used up.
- Prime with 4 IU (international units) every time you use a new nozzle and expel the dose. This will reduce air bubbles within the nozzle.
- To ensure effective insulin administration keep the top of the nozzle perpendicular (at an angle of 90°) relative to the skin administration site. After the injection, keep pressing the injector onto the administration site for at least 5 seconds.
- When drawing-up the insulin dose, take up 2 IU (international units) of additional insulin, to compensate for the removal of possible air bubbles. Tap the nozzle to remove any air bubbles and expel the additional 2 IU (international units) prior to administration to ensure correct dose is administered.

- Clean the injection site as directed by your Diabetes Care Health Provider. Avoid unnecessary contact with the Nozzle tip or the areas on the consumables that come into contact with the insulin. Always keep the nozzle cover and adaptor caps on between use. This will prevent possible contamination.

Device

- The InsuJet™ device can be used for up to 5,000 injections or the expiration shown on the packaging, whichever comes first.

Replacement Guidance	
Nr. of injections per day	Replace your Injector device:
3 or less	before the Use-by date of the device
4	41 months after first use <u>or</u> before device Use-by date*
5	32 months after first use <u>or</u> before device Use-by date*
6	27 months after first use <u>or</u> before device Use-by date*
7	23 months after first use <u>or</u> before device Use-by date*
8	20 months after first use <u>or</u> before device Use-by date*


* Whichever comes first


Disposal

- Device and consumables can be recycled where local facilities indicate or discarded in regular household waste.



For more information, please contact:

 Sol-Millennium Medical Inc.
311 S Wacker Dr. Suite 4100
Chicago, IL 60606

 888-385-1381
833-637-0857

 solm.com/insuJet
insuJet@solm.com

Exclusive Distributor



Please refer to the Instructions For Use for further elaboration and comprehensive details.

NACA-DC-682-0324 © Sol-Millennium Medical Inc., 2024. All rights reserved. InsuJet, NuGen and NuGen logotype are trademarks of European Pharma Group BV and NuGen Medical Devices, Inc. Sol-Millennium and Sol M logotype are trademarks of Sol-Millennium Medical Inc. Product images are not necessarily to scale. All product images are for illustration purposes only and may not be an exact representation of the product. Disclaimer: Some products may not be regulatory cleared/released for sale in all markets. Please contact the local Sol-Millennium sales office for current product assortment and availability. See Instructions For Use for full information, including indications, contraindications, warnings and precautions.