

InsuJet™ Storage and Maintenance



Storage


- Store the device inside the Carry Case.
- Store the InsuJet™ in dry conditions between 5°C and 40°C.
- Store your insulin according to the guidelines of the insulin manufacturer.
- Do not store the system in the refrigerator or in a damp area. Dampness may affect the device and cause corrosion.
- When you travel by plane or in the event of sudden pressure or temperature changes, keep the vial/cartridge adaptors in your hand luggage and upright, to prevent leakage of insulin from the vials.

Maintenance

- Handle the InsuJet™ with care. Do not drop the device or the consumables.
- Do not immerse the InsuJet™ in liquid.
- Place the Nozzle Cap on the Nozzle, after each administration.
- Place the cartridge/vial Adaptor Cover back on the cartridge/vial Adaptor, after drawing insulin into the Nozzle.
- Never try to repair a broken InsuJet™.

Disposal and Replacement

- All products can be discarded in regular household waste
- Discard the cartridge/vial Adaptor along with each empty insulin cartridge/vial.
- The InsuJet™ is a so-called 'active' medical device, meaning that its mechanical function will eventually deteriorate to a point that safe and comfortable administration can no longer be performed with it.
- Replace your InsuJet™ device in time. The same device should never be used for more than 5000 injections, or past the Use-by date.

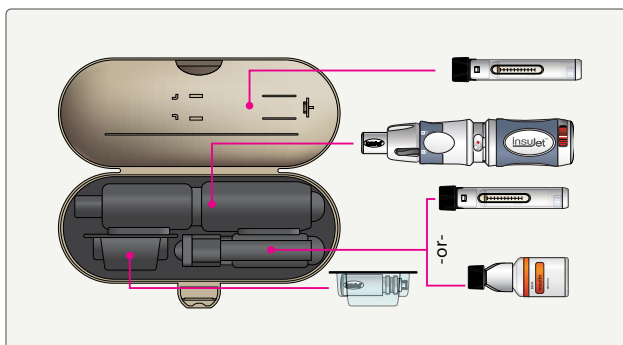



insuJet™

For more information, please contact:



Sol-Millennium Europe Sp. z o.o.
Warsaw HUB, Rondo Daszyńskiego 2B
00-843 Warsaw, Poland



solm.com/insujet
insujet@solm.com

nuGen
MEDICAL DEVICES

Exclusive Distributor

SolM Building
a Healthier
Tomorrow

EMEA-DC-660-0124 © Sol-Millennium Medical Group and Sol-Millennium Europe Sp. z o.o., 2023. All rights reserved. InsuJet, NuGen and NuGen logotype are trademarks of European Pharma Group BV and NuGen Medical Devices, Inc. Sol-Millennium and Sol M logotype are trademarks of Sol-Millennium Medical Group. Product images are not necessarily to scale. All product images are for illustration purposes only and may not be an exact representation of the product. Disclaimer: Some products may not be regulatory cleared/released for sale in all markets. Please contact the local Sol-Millennium sales office for current product assortment and availability. See Instructions For Use for full information, including indications, contraindications, warnings and precautions.

The content of this material is directed at and restricted to a professional audience only (Healthcare Professionals).

This material is distributed by Sol-Millennium Europe Sp z o.o. with a registered address at Twarda 18, 00-105 Warsaw, Poland, Registry Court: District Court for Warsaw, XIth Economic Department KRS 0000434783, EU VAT: PL5252535720, Share capital: 500.000 PLN.